



Patient Hook-Up Guide

7-Electrode, 3-Lead

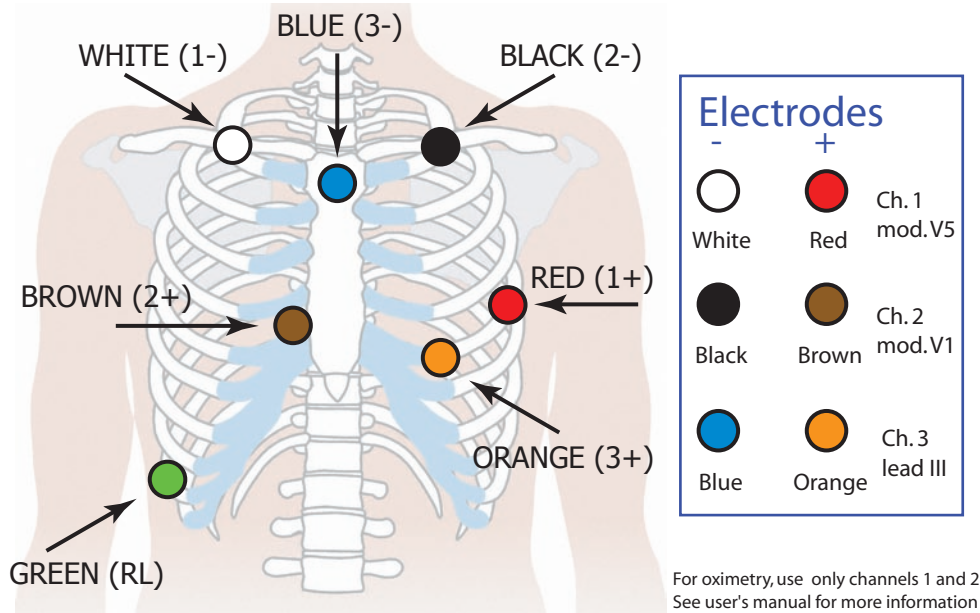


5-Electrode, 3-Lead

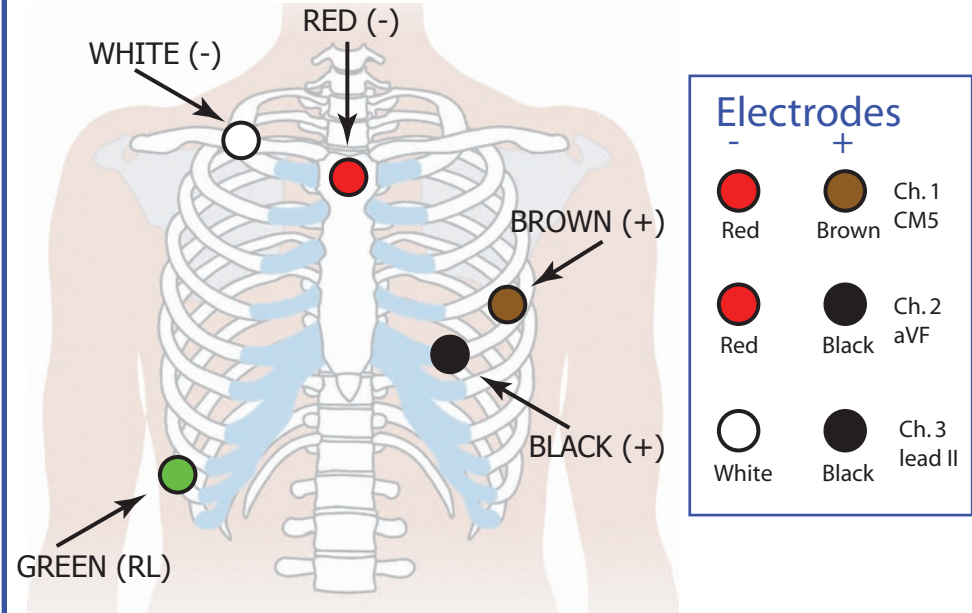

Skin Preparation


- ▶ Select area over bone, avoiding muscle areas and breast tissue.
 - ▶ Clean electrode sites aggressively with alcohol and use a razor to remove hair.
 - ▶ Abrade electrode sites with a scrub pad or gauze. Allow the sites to dry before applying electrodes.
 - ▶ Firmly snap electrodes to the lead wires, and apply to appropriate sites. Press the adhesive border firmly for consistent adhesion.
-
- Tape → [] ← Electrode
- ▶ To help alleviate tension, loop and tape lead wires down.

10-Electrode, 12-Lead

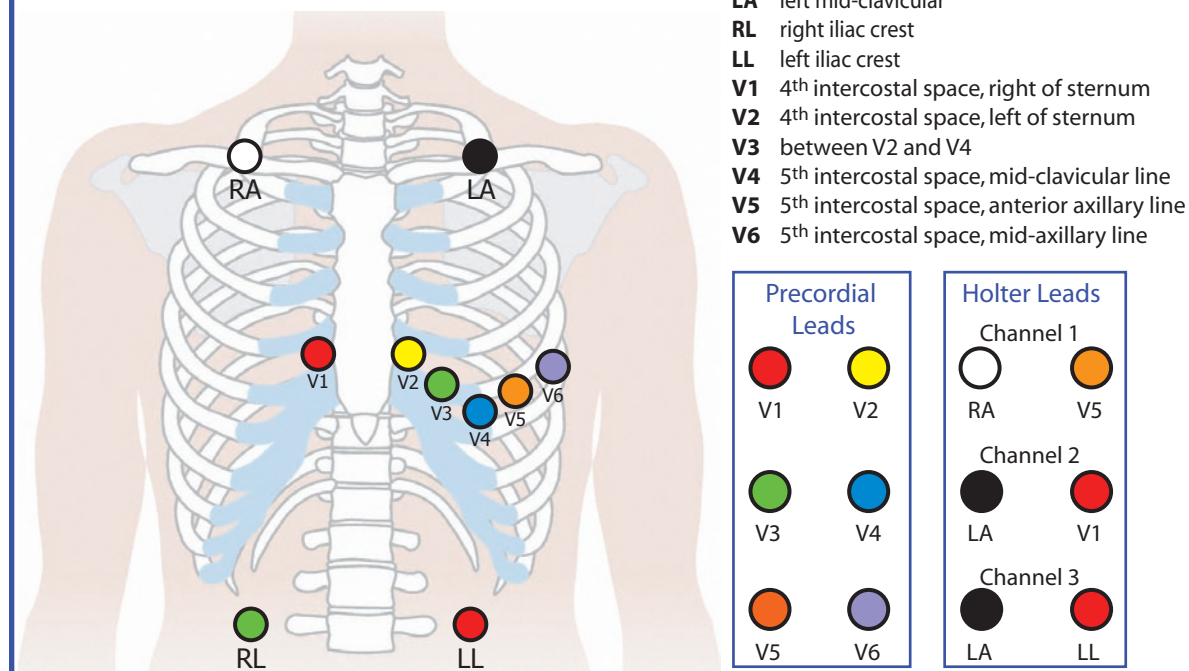

Quick DR180+ Setup Guide

Step 1

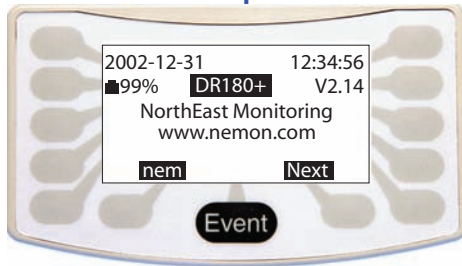


1. Insert the compact flash card into the recorder.
2. Insert 2 alkaline or NiMH batteries. Battery level should be at least 95%.
3. Press **Next**.

Step 5




1. *Lead Quality*, or impedance, should be 4 or 5.
2. Press **Next**.

Step 2

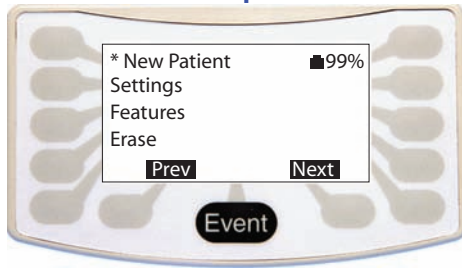


1. Press **Next** or **New Patient**.
- Press **Settings** to change the time, date, language, battery type, inverted display, daylight savings and mode lock.

Step 6

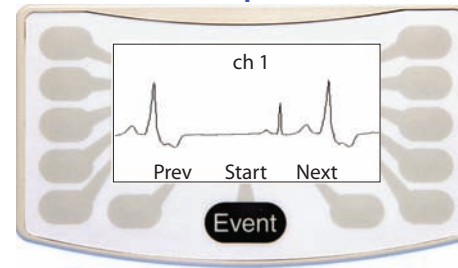

1. Press **Next** to cycle through the channels.
2. Press **Start** to begin recording.

Step 3

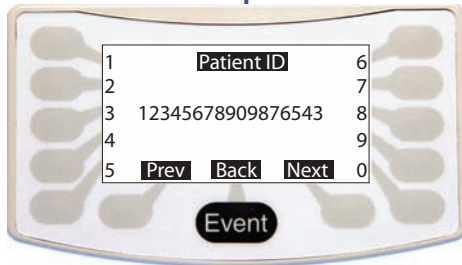

1. Enter *Patient ID* (up to 20 digits).
2. Press **Next**.


Step 7



DONE!

- See Step 2 to invert the display.

Step 4

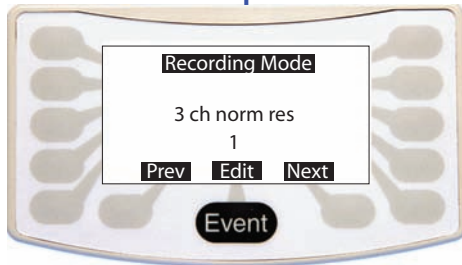

1. Press **Next**.
- For 12-lead and oximetry recorders, confirm the correct *Recording Mode*. Press **Edit** to change the mode. Also check for the correct cable.